





**CINIMEDIV**  
**SUNMEDIX**

# GREETINGS

## COMPANY

### ▶ GREETINGS

### SKIN STAPLER

### LAPAROSCOPIC PRODUCTS

### SPECIMEN RETRIEVAL BAG

### DVT PREVENTION PUMP

### EPIDURAL SPACE ANESTHESIA SENSOR

### CONTACT

We have started our business since 2004 from medical device developing. To realize the customer's idea, we have done from brainstorming to production, and the result is introducing some innovative products.

Since 2006, we have started our own products. Now, we are introducing some medical products and baby care products. Our philosophy of development and production is "User's convenience".

By trying our products, you can see the difference in design. Based on the customer's feed back, we are trying every day. You can see the result of our trying on our website.

It includes a skin stapler, laparoscopic devices and some innovative medical devices.

Now, we are starting the 2<sup>nd</sup> stage. Your advice and concern will be highly appreciated. Thank you.



# PRODUCT LINE-UP

SKIN STAPLER  
**ACOS™ SKIN STAPLER**



SKIN STAPLER  
**ACOS™ STAPLE REMOVER 1&2**



LAPAROSCOPIC INSTRUMENTS  
**VOLARGRIP™**



LAPAROSCOPIC INSTRUMENTS  
**G-COS**



SPECIMEN RETRIEVAL BAG  
**ONE POUCH™**



DVT PREVENTION PUMP  
**D-COS**



EPIDURAL SPACE ANESTHESIA SENSOR  
**ANES-Q**



# ACOS™ SKIN STAPLER ●○

## SKIN STAPLER

No doubt about your satisfaction with ACOS™ Skin Stapler



### ACOS SKIN STAPLER, AIMING TO BE THE BEST QUALITY PRODUCT

It is a single use skin stapler evaluated as less jamming, less empty shot with less power for operation. It can be applied to not only abdominal, gynecological, orthopedic and thoracic surgeries, but also for great variety of other skin closures. It could also be used for great variety of other skin closures.

#### FEATURES

- **Unique System** Unique ratchet system, preventing jamming
- **User-friendly** Ergonomic and user friendly design
- **Easy to Check** Easy to check remaining staples in cartridge
- **Clear View** Clear view on operating site



# ACOS™ SKIN STAPLER

SKIN STAPLER



COMPANY

SKIN STAPLER

▶ ACOS™ SKIN STAPLER

ACOS™ STAPLE  
REMOVER 1&2

LAPAROSCOPIC PRODUCTS

SPECIMEN  
RETRIEVAL BAG

DVT PREVENTION PUMP

EPIDURAL SPACE  
ANESTHESIA SENSOR

CONTACT

## DETAILS

- Materials** Medical-grade plastic, Stainless Steel
- Sterilization** ETO
- Conformity** CE
- Packaging** Individually packed in PET tray with tyvek

## ORDERING INFORMATION

Ref No.	Model Name	Description	Qty/Carton	Remark
0402A35W	Acos 35W	Acos disposable skin stapler, 35 wide staples	100 ea	-
0402A25W	Acos 25W	Acos disposable skin stapler, 25 wide staples	100 ea	-
0402A35R	Acos 35R	Acos disposable skin stapler, 35 regular staples	100 ea	-
0402A25R	Acos 25R	Acos disposable skin stapler, 25 regular staples	100 ea	-

## SPECIFICATION OF STAPLE

Type	Wide	Regular
Span (mm)	6.4	5.6
Height (mm)	4.0	3.8
Wire Diameter	0.57	0.57

# ACOS™ STAPLE REMOVER

SKIN STAPLER

No doubt about your satisfaction with ACOS™ Staple Remover



## FOR THE ACOS REMOVER, WE HAVE CONCENTRATED ON USER'S CONVENIENCE

We are using the same material of surgical blade for the safety. It can be operated with less power since the ergonomic design reduces the user's stress. All sizes of the staple are removable.

### FEATURES

- **Easy to Use** Easy to use with ergonomic design
- **Easy to Remove** Easy to remove staples with minimum force
- **Clear View** Clear view on operating site



# ACOS™ STAPLE REMOVER

SKIN STAPLER

COMPANY

SKIN STAPLER

ACOS™ SKIN STAPLER

▶ ACOS™ STAPLE  
REMOVER 1&2

LAPAROSCOPIC PRODUCTS

SPECIMEN  
RETRIEVAL BAG

DVT PREVENTION PUMP

EPIDURAL SPACE  
ANESTHESIA SENSOR

CONTACT

## DETAILS

<b>Materials</b>	100% Stainless Steel, or combined w/ plastics
<b>Sterilization</b>	ETO
<b>Conformity</b>	CE
<b>Packaging</b>	Individually packed in pouch

## ORDERING INFORMATION

Ref No.	Model Name	Description	Qty/Carton
0402ASR	Acos SR	Acos disposable staple remover	200 ea
0402ASR 2	Acos SR 2	Acos disposable staple remover	500 ea





# VOLARGRIP™ ●○○○○

LAPAROSCOPIC PRODUCTS

COMPANY

SKIN STAPLER

LAPAROSCOPIC PRODUCTS

▶ VOLARGRIP™


G-COS

SPECIMEN RETRIEVAL BAG

DVT PREVENTION PUMP

EPIDURAL SPACE ANESTHESIA SENSOR

CONTACT

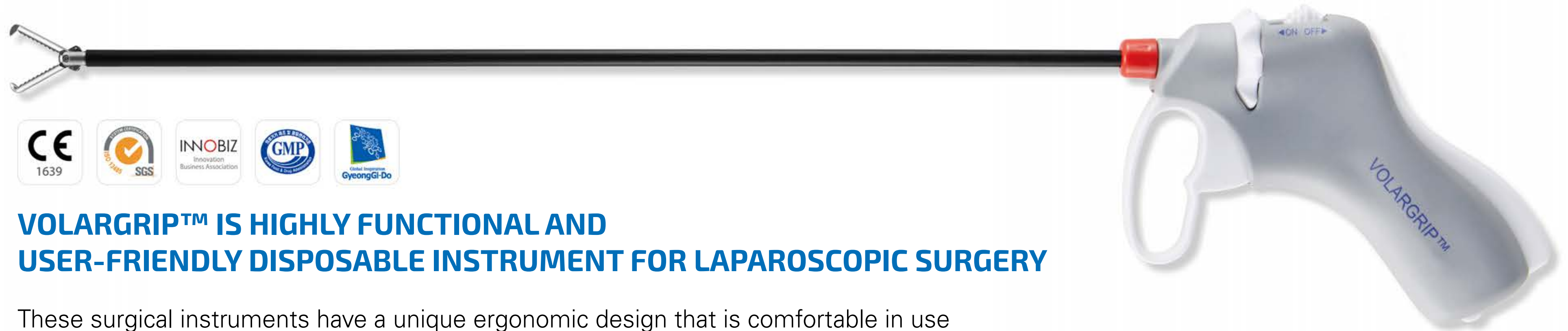

## VOLARGRIP™ IS HIGHLY FUNCTIONAL AND USER-FRIENDLY DISPOSABLE INSTRUMENT FOR LAPAROSCOPIC SURGERY

These surgical instruments have a unique ergonomic design that is comfortable in use and promotes efficient muscle action. The ergonomic handle eliminates deviation of the wrist and thereby reduces carpal tunnel pressure and eradicates fatigue and discomfort for the surgeon.



At least 7.5 million laparoscopic procedures are performed annually.



The increasing use of minimally invasive procedures is driven mainly by patient demands, as they are less traumatic, less painful, and require much less recovery time.

# VOLARGRIP™

LAPAROSCOPIC  
PRODUCTS

## LAPAROSCOPIC DEVICES FOCUSED ON USER'S CONVENIENCE

**With its unique design,  
the VOLARGRIP™ handle fits well in hand and has no distinct  
pressure areas because the handle contacts the hand at most.**



Rotation of the VOLARGRIP™ instruments is performed smoothly through a natural rotation of the wrist, and is also available through the easily accessible serrated dial. The patented VOLARGRIP™ handle is size-adjustable to suit different hand sizes.



For the surgeons, on the other hand, laparoscopic surgery is demanding due to longer duration, high precision demands, and increasing strain for hand and arm muscles.



VOLARGRIP™ gives the surgeon maximum comfort and enhances surgical precision.



This leads to a potentially positive clinical impact for the patient.

# VOLARGRIP™

LAPAROSCOPIC  
PRODUCTS

COMPANY

SKIN STAPLER

LAPAROSCOPIC PRODUCTS

▶ VOLARGRIP™

G-COS

SPECIMEN  
RETRIEVAL BAG

DVT PREVENTION PUMP

EPIDURAL SPACE  
ANESTHESIA SENSOR

CONTACT

## ERGONOMICS AND USE

### 1 Ergonomic Design




Pistol-shape designed handle promotes efficient muscle action, reducing carpal tunnel pressure.

### 2 360 degree Rotation




Jaw and shaft rotate 360 degree smoothly.

### 3 User-friendly Ratchet System

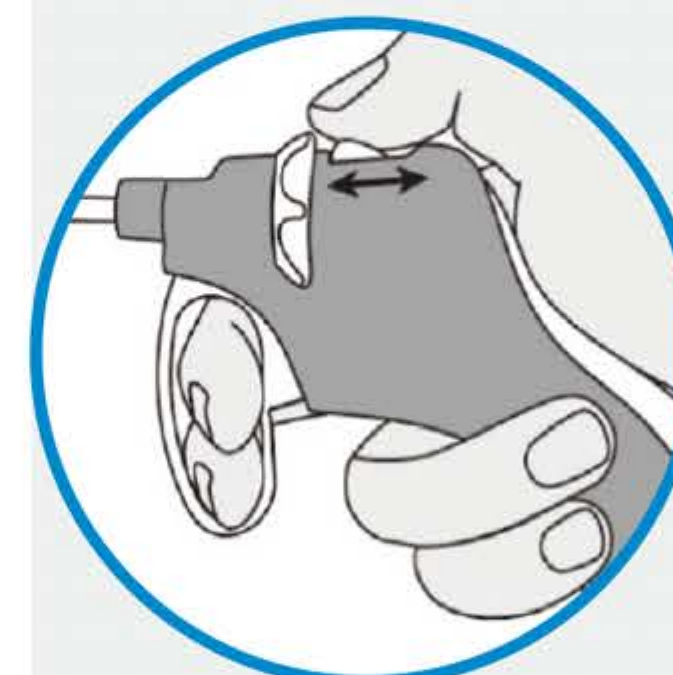


Volargrip Clinch and straight grasper have ratchet system which can be on and off for user's preference.

### 4 Size Adjustable Handle

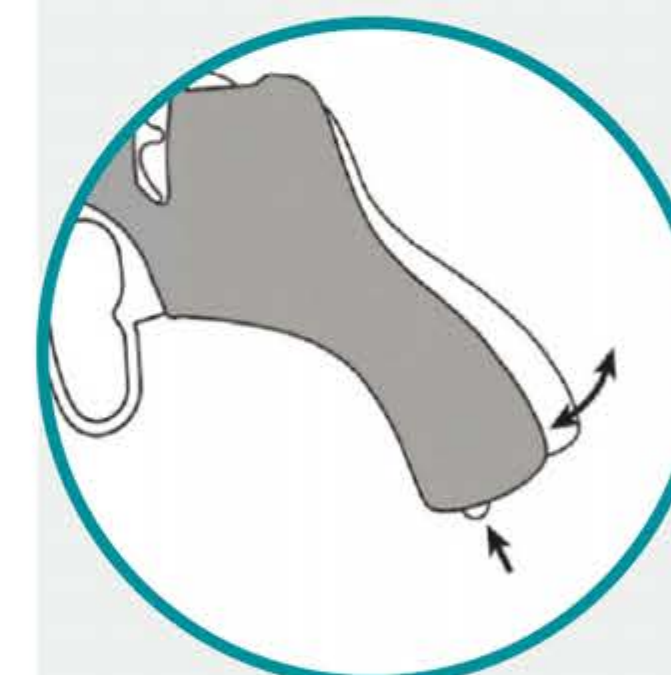

The patented size-adjustable handle fits well to all different sizes of hands.

### 5 Compatibility

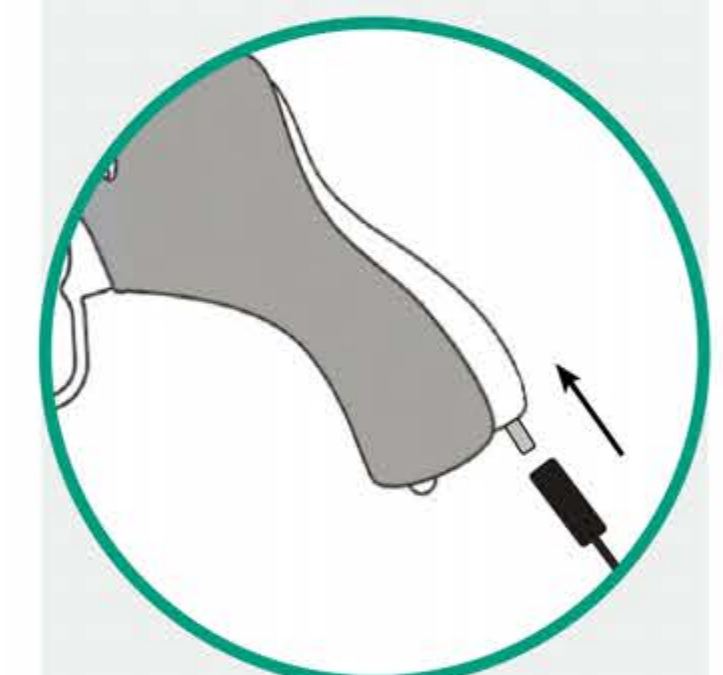

Volargrip instruments are applicable for all kinds of generators.

# VOLARGRIP™

LAPAROSCOPIC PRODUCTS

COMPANY

SKIN STAPLER

LAPAROSCOPIC PRODUCTS

**VOLARGRIP™**

G-COS

SPECIMEN RETRIEVAL BAG

DVT PREVENTION PUMP

EPIDURAL SPACE ANESTHESIA SENSOR

CONTACT

701-709

801-804/901

## ORDERING INFORMATION

Ref No.	VG30.701	VG30.702	VG30.703	VG30.704	VG30.705	VG30.706	VG30.707	VG30.708	VG30.709
Shape of Tip									
Description	Grasper, with hole	Grasper, with thin teeth	Grasper, with thick teeth - Clinch	Grasper, with thin teeth, rotating (20)	Grasper, with thin teeth, rotating (44)	Grasper, with wave teeth	Grasper, blunt	Grasper, cholecyst	Grasper, rat teeth
Color of Cap									
Q'ty/Carton	10EA								
Remark	Non-electronic								



# VOLARGRIP™

LAPAROSCOPIC PRODUCTS

COMPANY

SKIN STAPLER

LAPAROSCOPIC PRODUCTS

**VOLARGRIP™**

G-COS

SPECIMEN RETRIEVAL BAG

DVT PREVENTION PUMP

EPIDURAL SPACE ANESTHESIA SENSOR

CONTACT

701-709

801-804/901

Ref No.	VG30.801	VG30.802	VG30.803	VG30.804	VG30.901
Shape of Tip					
Description	Dissector, curved(Maryland)	Dissector, 90°	Scissor, Straight	Curved Scissors	Sealing and dividing device(Gcos)
Color of Cap					
Q'ty/Carton			10EA		
Remark	Monopolar				Bipolar / <b>New Item</b>

# G-COS

## LAPAROSCOPIC PRODUCTS

### Good for Laparoscopic Surgery Course

Reliable sealing and dividing device for single use.



### BENEFIT

1 Ergonomic Handle



2 360° Rotatable Shaft



3 Sealing Power Improved by Jaw Coating



4 Less Tissue Damage with Soft Cutting



# G-COS

## LAPAROSCOPIC PRODUCTS

### BENEFIT

- 5 Safety System for Blade Operation with Auto Lock
- 6 Applicable for All Kinds of Generators
- 7 Safe System for Cable Connector



### ORDERING INFORMATION

Ref No.	Shape of Tip	Color of Cap	Description	Qty/Carton	Remark
VG30.901			Sealing and dividing (straight) shaft length 330 mm, OD 5 mm	10	-

# ONE-POUCH™

## SPECIMEN RETRIEVAL BAG

### One-pouch Specimen Retrieval Bag



### SPECIMEN RETRIEVAL BAG : SIMPLE & EASY OPERATION

**For the specimen retrieval, it provides a simple, easy and efficient solution.**

One-pouch is designed to reduce the number of laparoscopic holes and allows the user to remove tissue and debris from the abdominal cavity safely and easily.

### FEATURES

#### Simple & Easy Operation

- It consists of a bag, cable and handle.
- The bag is inserted to the abdominal cavity through the trocar in use, while the handle stays on the outside.
- The user will navigate the specimen bag using a laparoscopic grasper.
- It is optimized for one port system and gynecologic surgery.

### ORDERING INFORMATION

Ref No.	Description	Q'ty/Carton
OP 350	One-pouch	100 ea
OP 500	One-pouch	100 ea
OP 700	One-pouch	100 ea



# ONE-POUCH™

## SPECIMEN RETRIEVAL BAG

COMPANY

SKIN STAPLER

LAPAROSCOPIC PRODUCTS

SPECIMEN RETRIEVAL BAG

▶ ONE POUCH™

DVT PREVENTION PUMP

EPIDURAL SPACE ANESTHESIA SENSOR

CONTACT

### HOW TO USE

- 1 While holding the cover pipe with a hand, carefully grasp the center of the specimen bag rim together with cable using non-traumatic laparoscopic grasper. Insert the specimen bag through the desired 10mm, 11mm or 12mm trocar cannulae.
- 2 Once the bag is inserted to the abdominal cavity, release the bag and remove the laparoscopic grasper.
- 3 Place the desired specimen in the bag and full out the handle placed on the outside while securing the bag with grasper to tie it.
- 4 Remove the trocar cannula backward. Pull out the handle until the mouth of the bag can be visualized through the incision site.
- 5 Upon completion of laparoscopic procedure, dispose this bag in accordance with local regulation. Refer to the above illustration.

# D-COS ●○

## DVT PREVENTION PUMP

### DVT Pump for active user

Portable pneumatic device for preventing Deep Vein Thrombosis for the user's convenience and wide application.



D-cos provides you with better solution to prevent DVT and save you from threatening side effects. You can simply wrap D-cos air cuff around your calf and turn on the device for your treatment. It delivers pneumatic pulse to your calf through air cuff and smoothly stimulates your blood circulation. You will be pleased with rhythmical air pressure.

### FEATURES

- **Lightweight and Portable**
- **Diverse Cuff Sizes for All Ages and Gender**
- **Disposable Cuff**

D-cos is portable DVT pump, Patients can be treated wherever they want at any time.

D-cos has 4 different sizes of cuff, providing all ages and gender with perfect fit cuffs.

As D-cos cuff is disposable, when the cuffs are contaminated, you can simply change cuffs only, you do not need to throw away the whole system, Save your cost!



COMPANY

SKIN STAPLER

LAPAROSCOPIC PRODUCTS

SPECIMEN  
RETRIEVAL BAG

DVT PREVENTION PUMP

D-COS

EPIDURAL SPACE  
ANESTHESIA SENSOR

CONTACT

# D-COS

## DVT PREVENTION PUMP

### ORDERING INFORMATION

Ref No.	Description	Q'ty / Carton	Remark
DC1505-03	Portable Pneumatic Controller	10	Set
CF03-WBSS	Cuff for calf - Extra small size	40	Pair
CF03-WBS	Cuff for calf - Small size	40	Pair
CF03-WBM	Cuff for calf - Medium size	40	Pair
CF03-WBL	Cuff for calf - Large size	40	Pair



# ANES-Q

## EPIDURAL SPACE ANESTHESIA SENSOR

### The Easiest Way to Find the Epidural Space

Do as you do. Just combine feeling and seeing.



It is not easy to find out where the epidural space is, but we can see and feel when we get there. You had concentrated the pressure on your fingers only, but now, you can surely feel and see.



### ORDERING INFORMATION

Ref No.	Description	Qty/Carton
Anes-Q	Pressure Sensor for Epidural Space	200

# ANES-Q

## EPIDURAL SPACE ANESTHESIA SENSOR

COMPANY

SKIN STAPLER

LAPAROSCOPIC PRODUCTS

SPECIMEN  
RETRIEVAL BAG

DVT PREVENTION PUMP

EPIDURAL SPACE  
ANESTHESIA SENSOR

▶ ANES-Q

CONTACT

### HOW TO USE

- 1** Check Appearance
- 2** Execute Manual Leakage Test
- 3** Inflate Balloon with Air or Saline
- 4** Advance the Needle until the Balloon Deflates
- 5** Inject Solution
- 6** Remove the Device

# CONTACT

SUNMEDIX

COMPANY

SKIN STAPLER


LAPAROSCOPIC PRODUCTS

SPECIMEN  
RETRIEVAL BAG

DVT PREVENTION PUMP

EPIDURAL SPACE  
ANESTHESIA SENSOR

CONTACT

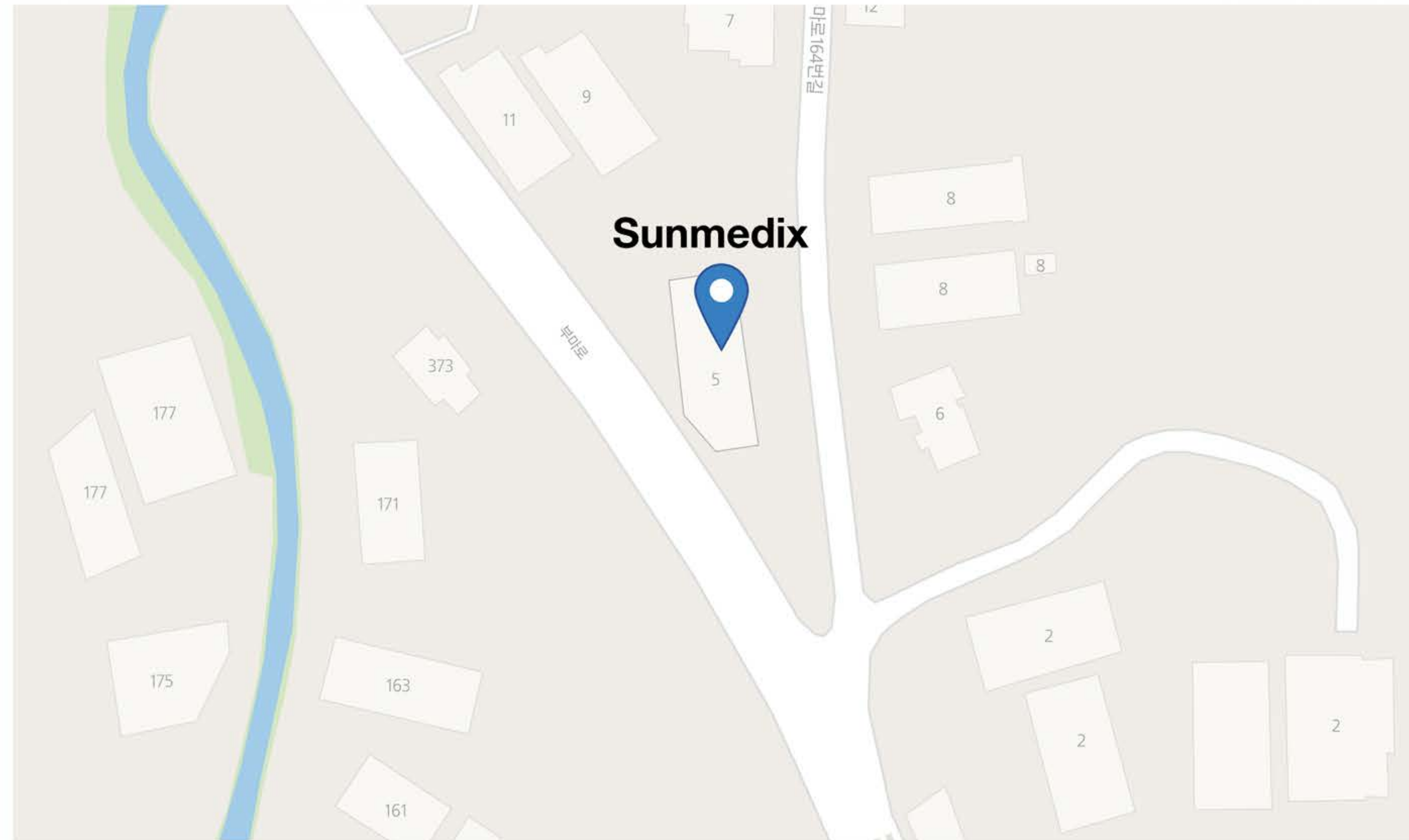

## Address

5, 9, 11, Buma-ro 164beon-gil, Jinjeop-eup, Namyangju-si, Gyeonggi-do, Republic of Korea 12000

## Tel

+82-31-574-7417

## Fax

+82-31-574-1599

## E-mail

[sales@sunmedix.com](mailto:sales@sunmedix.com)





5, 9, 11, Buma-ro 164beon-gil, Jinjeop-eup, Namyangju-si, Gyeonggi-do, Republic of Korea 12000

**T** +82-31-574-7417    **F** +82-31-574-1599    **E** [sales@sunmedix.com](mailto:sales@sunmedix.com)